

Authenticate The Brand You Trust

Since 1953 millions of people around the world have benefited from the excellent aesthetics, comfort and value that a Valplast® Flexible Partial Denture has brought to their smile. Many decades of innovation, experience and knowledge have allowed us to provide the best materials and product support to countless individuals around the world. *With so many new denture materials trying to make a name for themselves, how can you really know that Valplast® is being used to fill your prescription?**

All Valplast® Flexible Partial dentures should arrive at your practice with serialized Certificates. This provides traceability and disclosure of materials used, giving you assurance that your lab is using genuine Valplast® resin as you prescribe for your appliances.

Valplast International Corp. maintains an expert quality management system that is independently audited under ISO 13485 standards, giving you consistent quality medical devices that you can rely on.

Valplast® is the name you have always known and trusted. Accept no substitutes.

Look for the Valplast® Certificate of Authenticity with every Valplast® appliance that you prescribe.

"My Choice, for My Patients"



*Prescribe Valplast by BRAND on lab instructions.

valplast
Flexible Partial

Visit **Valplast.com**
Today!

- Register new Valplast® appliances at www.valplast.com/register
- Visit www.valplast.com/dentists1 for more information about Valplast® techniques
- Download Valplast® prescription forms at www.valplast.com/authentication

valplast
Compliance Services



Copyright Valplast International Corp. 2014
All Rights Reserved

Valplast International Corporation
200 Shames Drive
Westbury, NY 11590
(516) 442-3923 (Phone)
(800) 843-2861 (Toll-Free)
info@valplast.com (E-Mail)
www.valplast.com

valplast
Compliance Services



The **valplast**
Authentication Program



Authentication Program *Step-By-Step* Instruction Guide

What Your Lab Should Do

The serialized Valplast® Authenticity Certificates are distributed with every cartridge of Valplast® resin that your lab receives. Each Valplast® case from your lab should be accompanied by a completed Certificate of Authenticity.

1. The lab receives serialized Authenticity Certificates with each new package of Valplast® cartridges. They are retained to be completed with each Valplast® case they process.
2. When the lab is ready to process a new case, they select one of the Valplast® Authenticity Certificates and a cartridge of Valplast® resin. They record the material lot number from the cartridge in the space provided on the back of the certificate and place the certificate in the work pan accompanying the case. The lab is also required to record the lot number for their own traceability records. (Required in the USA, and recommended internationally.)
3. The lab processes the case as per your instructions. When the case is complete and ready for delivery, the lab writes their Certified Valplast Laboratory ID number* (optional) in the space provided on the back of the certificate and prepares the case for delivery.

**Certified Valplast Laboratory ID numbers are issued upon completion of the Valplast Certification exam. Laboratories are not required to become Certified Valplast Laboratories.*



Dental Office Procedure

Every Valplast® case that you prescribe should be accompanied by an original, uniquely serialized Certificate of Authenticity from the lab. Follow this simple guide to provide proof of material authenticity for every Valplast® appliance that you deliver to your patients.

4. Receive the new Valplast® case from your processing lab. The case should be accompanied by an original Valplast® Certificate of Authenticity with a unique serial number identifying the appliance as an authentic Valplast® restoration. Photocopies and scans are not considered valid.
5. **Prepare the case for insertion.** Immerse the Valplast® appliance in hot water for approximately one minute to bring it up to body temperature to allow for a smooth adaptation to the patient's mouth.
6. Check the fit of the appliance in the patient's mouth. If minor adjustments are necessary, use a Valplast® #13 Green Mounted Stone for gentle relief of the peripheral borders and tissue surfaces. Watch a video demonstration of our recommended adjustment procedure at youtube.com/valplastofficial
7. **Educate** your patient on how to care for their new Valplast® denture using **val-clean Concentrated Denture Cleanser**. Complete the information on the back of the Valplast® Authenticity Certificate and give to the patient. You may retain a copy for your records.



What Your Patients Should Do

Congratulations! Your patient is now the proud owner of a new Valplast® Flexible Partial Denture. All Valplast® appliances are guaranteed for life against breakage and fracture of the flexible denture base material. The Valplast® Certificate of Authenticity you give your patient provides proof of purchase and should be used by your patient to register their Valplast® appliance and to activate the warranty.

8. Your patient will need the Certificate to register their appliance. *This certificate contains a unique serial number that cannot be replicated and will verify the authenticity of the Valplast® material used to fabricate the denture.*
9. Your patient should be instructed to visit www.valplast.com/register to register the new Valplast® appliance and activate the warranty. If your patient cannot access the internet, you may register the appliance for your patient or instruct your patient to call us at **1-800-843-2861** to register.
10. Your patient should retain the Valplast® Certificate of Authenticity for their records and enjoy their beautiful new Valplast® smile!

